


# Jackson Medical Products LLC The "J" Clamp


## External Male Incontinence Clamp

The unique design of this clamp has a concavity in the upper portion of the frame which allows space for the blood flow while the clamp is being worn.

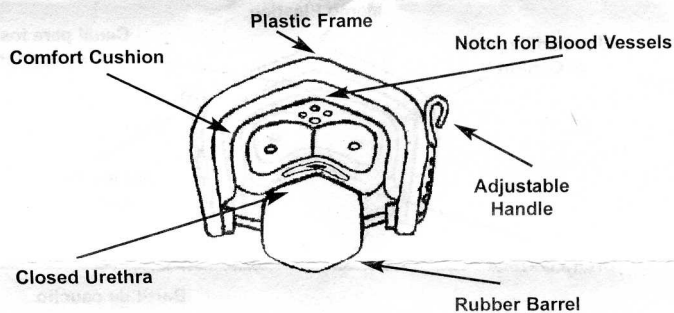

Please read and follow instructions for use:

The J Clamp is designed to adjust to most penis sizes. This is accomplished by opening the clamp by moving the adjustable handle counter-clockwise. Place the head of the penis through the opening with the adjustable handle on the right side of the penis (#1). The front of the J clamp should be against the back of the head of the penis. The plastic frame of "J" clamp should be on the shaft of the penis. Make sure the penis is centered in the clamp (#2). Hold the "J" clamp with the left hand and rotate the adjustable handle clockwise with the right hand until the barrel makes contact with the underside of the penis. Hold the penis up to see if the Rubber barrel is centered on the underside of the penis. (#3) This is important to make sure that the rubber barrel will compress the urethra to stop urine flow. Move the Adjustable handle clockwise until it engages the first notch on the frame. If you are able to urinate, move the handle to the next notch. Continue this process until you are unable to urinate. If you experience any discomfort, move handle counter-clockwise until desired setting. To remove the clamp, rotate handle counter-clockwise until the clamp is fully released. Urination is achieved when handle is released. Unlike other clamp devices, the "J" clamp is small enough to move through the zipper opening of pants.

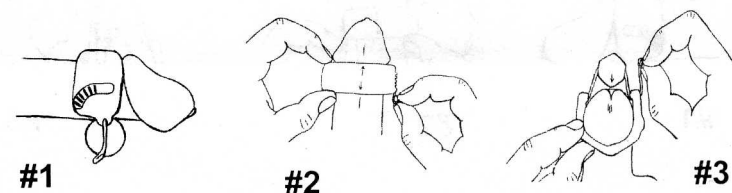

Care and Cleaning:

DO NOT SCRUB PRODUCT

Wash the clamp with mild soap and water and blot dry.

30 Day Guarantee

The "J" clamp is guaranteed to be free from defects. If there is a problem contact us immediately @ [www.jacksonmedicalproducts.com](http://www.jacksonmedicalproducts.com) or call 1-877-453-3531 or 615-453-3531

It is not guaranteed against normal wear or damage from abuse.

FDA Registered

patent pending